

READ THIS FIRST

Quick-Start Guide



SCAN THE QR CODE AT STEP 1
HERE TO VISIT THE
WEBSITE AND **SIGN UP**.

AFTER SIGNING UP,
LOG IN TO
CHB MEDICAL

CLICK ON 'TAKE A TEST'
AND THEN SCAN THE
QR CODE AT STEP 3
TO START YOUR TEST.

Taking the test

Technical Support :
info@fasterchecks.co.uk




First, TAKE A PHOTO of your blank test strip

This is your test strip BEFORE taking your test. PLEASE ENSURE THE IMAGE IS CLEAR AND SHOWS THE ENTIRE TEST STRIP. If you are using a laptop, hold up the blank test strip so it's clearly visible.



START YOUR VIDEO

When you're ready, press "START RECORDING" and conduct the test. Make sure we can see you in the video completing all the steps as per the instructions.

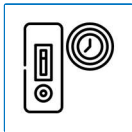

After 15 mins, TAKE A PHOTO of the result strip

Please ensure you complete this step as soon as the development time has elapsed, otherwise your submission may time out.



SELECT your test result

Please check that the result you select matches your test result.

WARNING

If you do not PERFORM THE TEST CORRECTLY, your test may be rejected.